

ATTENTION PHARMACISTS:
PLEASE POST THIS IN YOUR PHARMACY

Now Available!
Introducing KwikPen Prefilled with the Humalog® Brand of Insulins



Contact your wholesaler for normal ordering procedures

For use with all formulations of Humalog:

Humalog
KwikPen™
insulin lispro injection (rDNA origin)

Humalog^{mix}**75/25™**
KwikPen™
75% insulin lispro protamine suspension
25% insulin lispro injection (rDNA origin)

Humalog^{mix}**50/50™**
KwikPen™
50% insulin lispro protamine suspension
50% insulin lispro injection (rDNA origin)

How does KwikPen fit in?

- Small size
- Lightweight and portable
- Short thumb-reach at high doses
- Easy to set the dose*¹
- Easy to dial up and back down for dose correction
- Easy to dispense the maximum dose of 60 units*¹
- Does not need refrigeration after first use

*KwikPen Design Validation User Study included adult male and female participants with type 1 and type 2 diabetes. Of the total 150 study participants, 56 were insulin naïve, 42 were currently administering insulin with a vial and syringe and 52 were experienced insulin pen users.

¹ Data on file (HUM20071024A)

Please visit www.Humalog.com today to view:

- A demonstration on the use of KwikPen and full user manual
- Prescribing information on the Humalog Brand of Insulins
- Downloadable patient educational materials in multiple languages

Or contact The Lilly Answers Center at 800-Lilly-Rx (800-545-5979)

KwikPen prefilled with the Humalog Brand of Insulins will be color coded.

Humalog®, Humalog® Mix75/25™ and Humalog® Mix50/50™
are also available by prescription in the original Humalog and Humalog Mixtures Prefilled Pen.



Lilly

**Please see Important Safety Information on back and
accompanying full Prescribing Information**

	NDC, UPC, & Lilly Product Codes	Box of 5 Pens: Dimensions	Box of 5 Pens: Weight
Humalog KwikPen Prefilled insulin delivery device, 3.0 ml (Box of 5 pens)	NDC: 0002-8799-59 UPC: 3-0002-8799-59-8 LILLY: HP8799	5.8 in x 3.6 in x 1.2 in	6.6 oz
Humalog MIX75/25 KwikPen, Prefilled insulin delivery device, 3.0 ml (Box of 5 pens)	NDC: 0002-8797-59 UPC: 3-0002-8797-59-4 LILLY: HP8797	5.8 in x 3.6 in x 1.2 in	6.6 oz
Humalog MIX50/50 KwikPen, Prefilled insulin delivery device, 3.0 ml (Box of 5 pens)	NDC: 0002-8798-59 UPC: 3-0002-8798-59-1 LILLY: HP8798	5.8 in x 3.6 in x 1.2 in	6.6 oz

Humalog®, Humalog® Mix75/25™ and Humalog® Mix50/50™ are also available by prescription in the original Humalog and Humalog Mixtures Prefilled Pen.

Humalog insulins are for use in patients with diabetes mellitus for the control of hyperglycemia. Humalog should be used with longer-acting insulin, except when used in combination with sulfonylureas in patients with type 2 diabetes.

Important Safety Information

Humalog insulins are contraindicated during episodes of hypoglycemia and in patients sensitive to Humalog or one of its excipients. The safety and effectiveness of Humalog in patients less than 3 years of age have not been established. Safety and effectiveness of Humalog Mix75/25 and Humalog Mix50/50 in patients less than 18 years of age have not been established. There are no adequate and well-controlled clinical studies of the use of Humalog in pregnancy or nursing mothers. Potential side effects associated with the use of all insulins include hypoglycemia, weight gain, hypokalemia, lipodystrophy, and hypersensitivity. Because of the difference in action of Humalog, care should be taken in patients in whom these conditions may be clinically relevant (eg, those who are fasting, have autonomic neuropathy or renal impairment, are using potassium-lowering drugs, or taking drugs sensitive to serum potassium level). **Patients should be advised not to mix Humalog Mix75/25 or Humalog Mix50/50 with another insulin. Starting or changing insulin therapy should be done cautiously and only under medical supervision.**

Please see accompanying full Prescribing Information.



Humalog®, Humalog® Mix75/25™ and Humalog® Mix50/50™ are registered trademarks of Eli Lilly and Company. Humalog® KwikPen™, Humalog® Mix75/25 KwikPen™ and Humalog® Mix50/50 KwikPen™ are trademarks of Eli Lilly and Company.

Humalog® KwikPen™, Humalog® Mix75/25 KwikPen™ and Humalog® Mix50/50 KwikPen™ are available by prescription only.

HI49006-A © 2007 ELI LILLY AND COMPANY. PRINTED IN USA. ALL RIGHTS RESERVED.

Humalog is a registered trademark of Eli Lilly and Company.
Humalog is available by prescription only.

KwikPen is a registered trademark of Eli Lilly and Company.
KwikPen is available by prescription only.

HI-XXXXX XXXX 2007 ELI LILLY AND COMPANY. PRINTED IN USA. ALL RIGHTS RESERVED.